

How to Use

Dose & Time of Irradiation

Generally, the dose given is in Joule. The actual dose given will depend on the condition treated. Time will vary from 30 sec. to few minutes for each complete treatment session.

Frequency of Irradiation

Treatment is given once daily generally till healing & may be continued further at intervals of several days.

Comprehensive Treatment Approach Treat two or more of following

- * The entire injured or diseased area
- * Any Trigger or Tender Points
- * Any referred areas
- * Nerve Root or Superficial Nerve Trunks
- * Acupuncture / Auricular Points

CE Certified Laser safety glasses

Comfortable frame
Assorted Wavelengths



Contraindications

Direct Irradiation of the eye
Pregnant Patients
Over Thyroid Gland
Over areas recently injected with steroids or anti inflammatories
Where masking pain management maybe harmful
Incase of pace maker present within the area

Laser Classification :

Class IIIB & Class 2M

Wavelength :

635 nm, 658 nm,
660 nm, 808 nm,

O/P Power :

8mW, 12mW, 20mW, 22mW,
30mW, 36mW, 50mW, 60mW
80mw, 90mW, 100mW,.....

- Compact Laser - CL - mini • CL - plus • CL - extra
- Blood Irradiation Laser - BIL
- Infra Red - IR

Prof. Silberbauer
is a member of



These instruments meet the following standards:
EN 60601-1:2007
EN 60601-2-22:1996
EN 60825-1 + A1 + A2:2003
EN 60601-1-2:2007

The company Silberbauer had been certified by SGS United Kingdom Ltd. (Notified Body No. 0120), office for the certification of medical products, according to regulation 93 / 42 / EWG annex II, in agreement with the ISO 9001:2008 and ISO 13485:2003.

Silberbauer India


24-25, Prabhakar, Bhelke Nagar, Kothrud, Pune - 411029. (MS) INDIA.

Ph. : +91 - 20 - 65213282 Fax : +91 - 020 - 25387864
Cell : + 91- 9823070149 / + 91- 9823170149

Web : www.silberbauer.at/india | india@silberbauer.at
Mail : www.mylaser.in | team@mylaser.in



Authorised Dealer

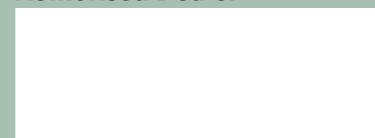

Silberbauer INDIA



Wound healing
Infection Control
Pain Management
Faster & Sure !



www.mylaser.in

NON-INVASIVE



About Silberbauer

In Vienna, Austria, Europe, there is the Laboratory Silberbauer, Medical Electronics. Founded in 1978 by Dipl.Ing. Gerhard Silberbauer. The company has the intention to explore the diagnostic and therapeutic possibilities in medicine with complementary & holistic methods. In the year 1979, the company developed and produced the Soft Laser in Austria and one of the first ones in Europe.

What is Compact Laser ?

It is a brand owned by Silberbauer, who is Producing Low Level Laser Therapy (LLLT) equipments. Compact Lasers have been permanently evolved & improved, used & exported to various countries all over the world during last 30 years.

Applications

General Practice is very important branch of Medicine where 'Compact Laser' is a boon in practice. The Therapy is used in Europe since last more than 30 years & in 2002 also approved by FDA in US. Today in India, Several practitioners from various discipline such as Ayurveda, Homeopathy, Alopthy, Dentistry, Physiotherapy, Acupuncture, Naturopathy, Yoga Therapist are using Compact Laser successfully.

It is of great benefit to today's General / Family Practice because of :

- Low cost treatment
- No special set up is required
- No/Low side effects
- Low on Investment, High on Returns

Common Indications in General Practice

- ★ Bone and Joint problems viz: joint pain relief
- ★ Osteoarthritis of knee, hip
- ★ Spondylosis of cervical & lumbar region
- ★ Ligament injuries of small to moderate severity
- ★ Sprains, Strains & swelling due to falls, trauma
- ★ Traumatic accidental injuries & wounds
- ★ Minors burns & scaled injuries
- ★ Oral cavity problems viz:
- ★ Aphthous Stomatitis & Ulcers
- ★ Gingivitis
- ★ Pain of dental caries
- ★ Angular Stomatitis
- ★ Herpes Simplex (oral & genital)
- ★ Herpes Zoster
- ★ Allergic Rhinitis
- ★ Tonsillitis, Pharyngitis
- ★ Sinusitis
- ★ External Otitis
- ★ Simple undisplaced fractures of phalanges, ribs
- ★ Pediatric & geriatric cuts and wounds
- ★ Non - healing wounds of various types
- ★ Bed sores
- ★ Simple cuts & wounds in household work
- ★ Fissure
- ★ Improve sperm count
- ★ Premature hair loss



Features

- No Consumables, No Disposables
- Solid state (Diode) Technology
- Dual Power Operation - Battery / Mains
- European Medicals Grade - Safety Standard's
- O/p corrections + 1% by Hidden Monitor
- Micro-Controller based
- Automatic dose regulation
- Painless Treatment
- Continuous Wave



Compact laser treatment can be used wherever we need to -

- Accelerate wound healing
- Control of Inflammation
- Infection Control
- Pain Reduction

Laser Acupuncture

- Body & Ear
- No prickling pain
- Better results with children
- No danger of infection
- Specially useful for points in the area of lesions
- Special indications (e.g. trigeminal neuralgia) You may use a combination of body acupuncture or ear acupuncture with laser.

Laser Blood Irradiation (Intravenous Non Invasive)

- *The treatment possibilities of the method are very large and include the improvement of rheological characteristics of the blood and microcirculation, normalization of parameters of hormonal, immune, reproductive and many other systems.
- *IV LBI is used for its biostimulative, analgetic, antiallergic, immunocorrective, antitoxic, vasodilative, antiarrhythmic, antibacterial, antihypoxic, spasmolytic, anti-inflammatory and some other properties.
- * Promotes improving the rheological properties of blood, rising fluidity and activating transport functions.

